
# HEALTH MASK

## MADE IN ITALY

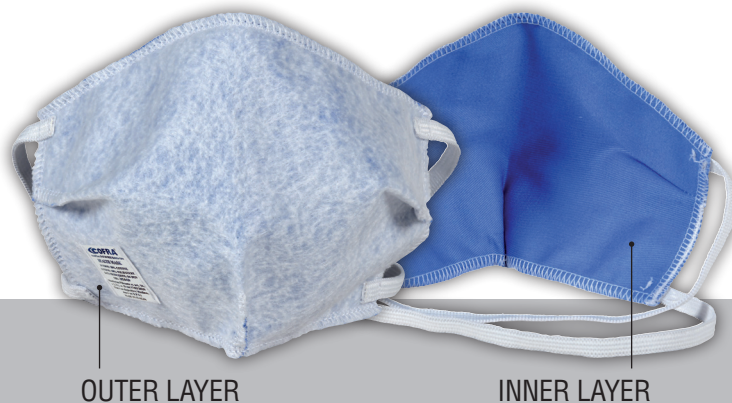OUTER LAYER

INNER LAYER

UK

| MATERIALS   |                                                                      |  |
|-------------|----------------------------------------------------------------------|--|
| OUTER LAYER | 100% water repellent recycled polyester (110 g/m <sup>2</sup> ± 10%) |  |
| INNER LAYER | 100% cotton (190 g/m <sup>2</sup> ± 10%)                             |  |
| ELASTICS    | Spandex and nylon                                                    |  |

| TECHNICAL SPECIFICATIONS         |                               |                       |                         |
|----------------------------------|-------------------------------|-----------------------|-------------------------|
| Test                             |                               | User's protection (*) | Droplet retention (**)  |
| AIR PERMEABILITY (BREATHABILITY) | Pressure difference of 100 Pa | -                     | 183 L/m <sup>2</sup> ·s |
| PROTECTION AGAINST AEROSOLS      | Particles of 3 µm diameter    | 98%                   | 98%                     |
|                                  | Particles of 1 µm diameter    | 96%                   | 97%                     |
| WATERPROOFNESS (outer layer)     | Water dragging                | 0 mm                  |                         |

(\*) Flow measured from the outside towards the inside, during inhalation.  
(\*\*) Flow measured from the inside towards the outside, during exhalation.

|                    |          |                                                                                                       |
|--------------------|----------|-------------------------------------------------------------------------------------------------------|
| SIZE               | One size |                                                                                                       |
| SINGLE PACKED MASK |          | The single packaging improves hygiene by avoiding any contact with harmful substances before the use. |

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1. Wash hands and face with soap before and after using the mask and before touching it.
2. Wash mask before use and before wearing it.
3. Take the mask from the external part, put the mask on the face covering your nose, your mouth and your chin; be sure the elastics are well placed behind your head.
4. After putting the mask properly, do not touch the mask with your hands.

## Filtering Mask REUSABLE

**MAXIMUM TEMPERATURE: 60 °C**  
(10 WASHING CYCLES ARE GUARANTEED).  
FOR A LONGER LIFE OF THE PRODUCT,  
GENTLY HAND WASH AT 35-40 °C

**DO NOT IRON**

|      |                                                                         |
|------|-------------------------------------------------------------------------|
| CODE | M031-B100 BOX containing 100 pcs.                                       |
|      | M031-K100 CARTON containing 1000 pcs.<br>(10 boxes containing 100 pcs.) |

- Pursuant to articles 102-113 of the "Consumer Code", Legislative Decree 6 September 2005, n. 206, declares that this product is neither a PPE nor a Medical Device; does not harm people's health.
- Pursuant to paragraph 2 of article 16 of Legislative Decree of 17 March 2020, n. 18, and of circular of the Ministry of Health of 18 March 2020, to contain the spread of the COVID-19 virus and until the end of the emergency state mentioned in the resolution of the Council of Ministers on January 31, 2020, individuals present on the national territory are authorized, as a precaution, to use the filter masks without the CE mark and produced notwithstanding the current marketing regulations, which for their destination are not considered neither as Medical Device nor as PPEs.
- Pursuant to paragraph 2 of article 16 of Legislative Decree of 17 March 2020, n. 18, the assessment procedures do not apply referred to in Article 15 of Legislative Decree March 17, 2020, n. 18.

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by Cofra S.r.l.
Emis par Cofra S.r.l.
Emitted por Cofra S.r.l.